



USER MANUAL

**Pro 9000**

Teeth Whitening Lamp

**Celebrity Whitening**

## **AFTER SALES SERVICE AND SUPPORT**

Please read this user manual before using the PRO 9000 to ensure correct settings are used.

If you experience difficulties with your Pro 9000 please contact us for after sales support and service:-

Email | [smile@celebritywhitening.com.au](mailto:smile@celebritywhitening.com.au)

Telephone | +61 7 55 11 8216.

## Contents

<b>Chapter 1: At a glance</b>	<b>5</b>
Overview	5
Accessories	6
Pro 9000 features and benefits	7
Technical specifications	7
Product components	8
<b>Chapter 2: Getting started</b>	<b>9</b>
Steps to Assemble lamp	9
steps to connect power switch	10
Lamp operation during teeth whitening	11
<b>Chapter 3: Warnings</b>	<b>13</b>
In case of emergency	13
<b>Chapter 4: Frequently asked questions</b>	<b>15</b>
<b>Appendices</b>	<b>25</b>
Appendix 1 – troubleshooting guide	25
Appendix 2 – taking care of your lamp	25
Appendix 3 – teeth whitening procedure guide	26

This page has been intentionally left blank

## Chapter 1: At a glance

### OVERVIEW

The Pro 9000 LED cold accelerator lamp is one of the most advanced teeth whitening lamps available in Australia today. The lamp is ideal for salons and dentists. When used in conjunction with our teeth whitening gels; and after completing our complimentary training will remove stubborn stains from the average person's teeth and a person with heavily stained teeth.

### Why use and accelerator lamp?

Using a LED teeth whitening machine accelerates the teeth whitening gel, causing it to react faster when breaking down stains. What this means for your client is what normally takes around 14 hours to achieve with whitening gel alone they can achieve in a one hour in-chair treatment. The use of the accelerator lamp speeds up the process of the molecular breakdown of the whitening gel which makes the oxidization of the stains occur more rapidly. Your client will get several benefits with the lamp process including:-

- only needing to stick to a white diet once, rather than over several weeks
- getting instant results, while clients using home treatments often get bored with the process before they attain maximum results.

## ACCESSORIES

Included with your Pro 9000 is:-

<i>Item name</i>	<i>Number of pieces</i>
Lamp head	1
Rotational arm	1
Top arm with handle	2
Base pole	1
Legs	5
Decorative Ring	2
A Connector	1
Allen key	1
Hex screw	1
Specifications	1
Blinker	2

## PRO 9000 FEATURES AND BENEFITS

- services back to back clients each day.
- ideal for servicing clients with normal to *heavily* stained teeth such as smokers
- Start the lamp on 100% each day and if necessary reduce to 80% after you first two directly consecutive clients
- has 40% more light intensity than the Force 6000. *The light intensity on the PRO 9000 is the maximum you need to get the best possible result in combination with the whitening gels and your expertise*

## TECHNICAL SPECIFICATIONS

<b>Dimension</b>	Height: 1420mm, Arm length: 1250mm (max), Wide: 630mm
<b>Weight</b>	18.5kg (requires a dedicated area)
<b>Input</b>	AC100 to 240V, 5060Hz
<b>Output</b>	4.7V
<b>Power</b>	40VA
<b>Wavelength</b>	The blue light wave length is 460 to 490mm.
<b>Density of</b>	200 to 300mw/cm <sup>2</sup>
<b>Whiten time</b>	Recommend: 3 x 20 minutes sessions per treatment
<b>One year</b>	<a href="https://www.celebritywhitening.com.au/index.php?main_page=shippinginfo">https://www.celebritywhitening.com.au/index.php?main_page=shippinginfo</a>

## PRODUCT COMPONENTS

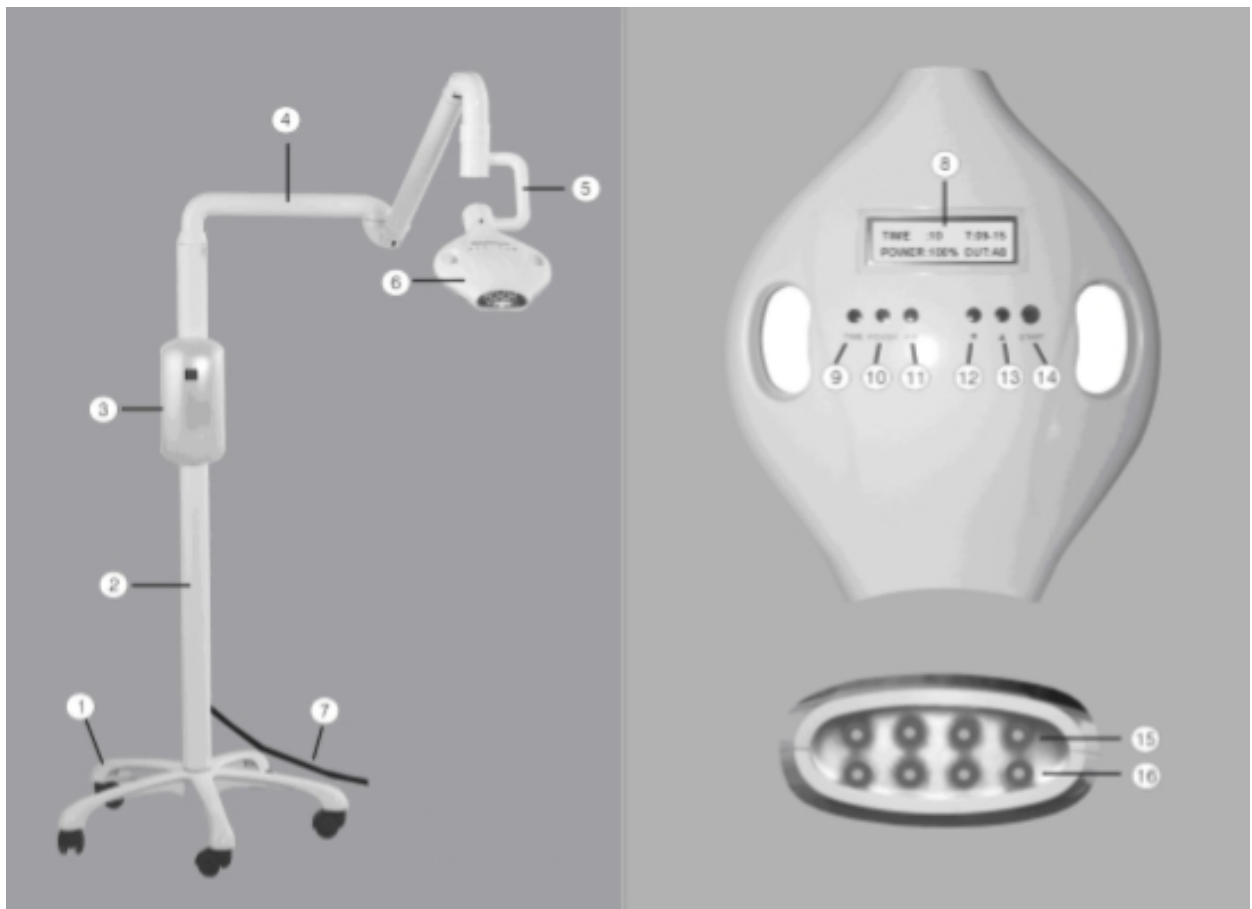


Fig 1. Structure and components

1.	Legs	2.	Base pole	3.	Power switch
4.	Rotational arm	5.	Top arm with handle	6.	Lamp head
7.	Power wire	8.	Screen	9.	Time button
10.	Power button	11.	A/B Group button	12.	Down button
13.	Up button	14.	On/off button	15.	LED Lights (Group A)
16.	LED Lights (Group A)				




## Chapter 2: Getting started

### STEPS TO ASSEMBLE LAMP



1 Install all 5 legs onto the horse.

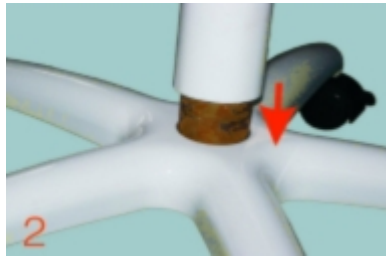

2 Insert the base trunk pole into the legs.



3 Secure pole and legs together by placing screw on stud and tighten clockwise with a wrench.



4 Screw both base and upper poles together.




5 Insert the inner stainless steel arm into the rotational white arm and secure by twisting in a clockwise direction.



6 Connect the plug of the stainless steel arm and the inner socket together.

Force 6000 User Manual

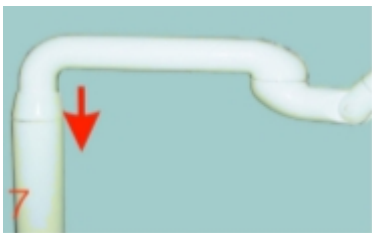

7 Connect the white rotational arm into the inner white arm stud.



8 Connect the white rotational arm into the inner stud.




9 Screw the white arm into the inner stud of the rotational arm with screwdriver provided.

## STEPS TO CONNECT POWER SWITCH



1 Drill a hole in the pole where you want to attach the power switch.

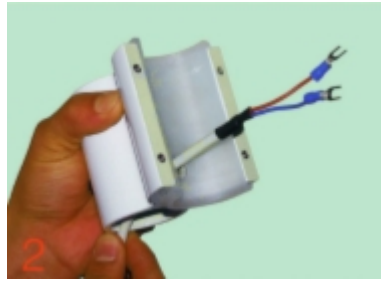

2 Position power switch bracket on base pole and feed wiring through the drilled hole.




3 Push the wiring into the drilled hole until it is long enough to reach the connection port of the dental unit.




4 Lean the power bracket against the pillar ensuring 6cm of wiring is left out and that the wiring is not pinned down by the power bracket.



5 Screw the back part of the bracket to the front part of the bracket.

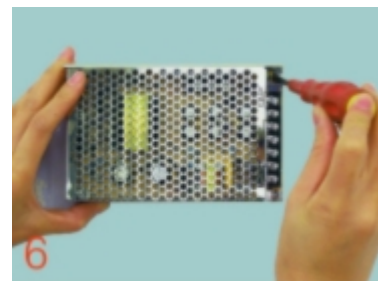


6 Fix the power supply in the correct position in the lamp head.

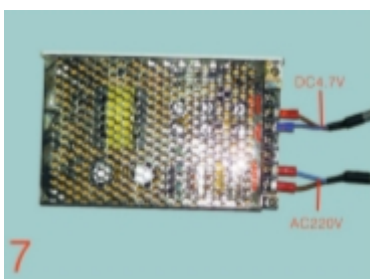

1. Connect the power:-
  - Blue port to [COM]
  - Red port to [V+]
2. Connect the two red ports of the 90cm wire to the [LN] (AC220V) interface on power supply.



8 Screw the stainless connector into the arm and then place the connector into the bracket.

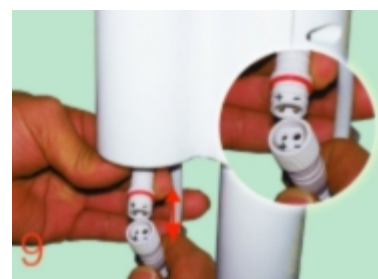

9 Connect the plug on the arm (with handle).



Connect to the handle.



Plug the lamp head into the arm and fix with screw driver supplied.

### LAMP OPERATION DURING TEETH WHITENING

1. Plug the lamp into the power socket and turn on the power.
1. Press **[TIME]** to set time usage with ▲ or ▼ buttons. Each press of the button to increase or decrease the time by 1 minute. You can set a maximum timer for 30 minutes.
2. Press **[POWER]** to adjust the light magnitude using ▲ or ▼ buttons. We recommend starting on 100% and then reducing the magnitude to 80% after your first two clients each day. You will *still* achieve the best possible results for your client with the lamp at 80%. To default the setting to 100% press **[POWER]** button twice.
3. Press **[A/B]** to set up the output required using ▲ or ▼ buttons. There are three modes to choose:-
  - **[A]** for the top row of teeth
  - **[B]** for the bottom row of teeth
  - **[AB]** for both top and bottom teeth.
4. Press **[START]** to turn on the lamp. The menu on-screen will light up and show the default working time. When the time is up the lamp will turn off automatically. If you need to stop the lamp manually then press **[START]** (Note. When the lamp stops working the cooling fan inside the machine will go on working for another minute to protect the LED pipe and the safety system before stopping).
5. There is a thermometer guard unit in the lamp. When the temperature of the blue light LED pipe exceeds 60°C the lamp will stop working to protect the safety system.

This page has been intentionally left blank

## Chapter 3: Warnings

1. Use this device only for teeth whitening treatments
2. Sterilise the lamp prior to use.
3. Do not stare at the lamp directly when in operation. Ensure both the operator and clients are wearing protective glasses at all times when the machine is turned on to avoid harming the eyes.
4. When moving the lamp ensure you support the rotational arm with lamp in one hand and the stem of the lamp base with the other. Ensure that the lamp does not tip over and that the tripod does not move.
5. Store the lamp in -10 to 40°C and ensure no corrosion occurs on the lamp. When not using the lamp ensure lamp head is facing downwards and put the protective jacked over the lens to ensure no dust, wax, hair, gel or fine particles get into the circuit board.
6. Your lamp has been thoroughly tested and is fully compliant with all Australian regulations and legislation. Please do not disassemble the light head or the control system without speaking with one of our experienced staff or consulting an electrical engineer.
7. Teeth whitening is NOT for the following people:-
8. children under 16 years of age
9. women who are pregnant or breastfeeding
10. people allergic to hydrogen peroxide
11. people with gum disease
12. people requiring dental work

### IN CASE OF EMERGENCY

If there is an emergency press **[START]** and this will stop the lamp immediately.

This page has been intentionally left blank

## Chapter 4: Frequently asked questions

Q What is teeth whitening?

A Teeth whitening restores the natural colour of the teeth by removing stains that have built up by lifestyle and diet.

Q Why do we need teeth whitening?

A As a person ages the adult teeth often become darker due to changes in the mineral structure of the teeth as the enamel becomes less porous. Teeth become stained by bacterial pigments, foods and beverages.

Teeth whitening expectations can be confusing when you see models on TV endorsing teeth whitening products who actually have veneers. Unfortunately there are also many results on social media that use photo filters and are not teeth whitening actual results. Teeth whitening businesses only share their very best results on social media so you are not seeing the results that are average or not so great. All of this can cause many clients to feel disappointed with their results.

Technique is as important as product. Some clients don't respond well for a variety of reasons including age, enamel condition and type of discoloration.

Q How safe is teeth whitening?

A We offer the latest whitening gel approved for use in Australia. We have a hygienic non-touch procedure. At NO time can you place your fingers in your client's mouth. Our gels are gentle and safe, problems are extremely rare.

Q Do you have the right expectations?

A Teeth Whitening does not give the client paper white teeth regardless of the level of peroxide used unless their teeth were that colour prior to staining. Movie star white teeth are often a result of porcelain veneers not teeth whitening. This does make it hard when there are ads on TV with models advertising teeth whitening products who don't have natural teeth.

It is an important part of the teeth whitening process to ensure your client has the right expectations prior to having a teeth whitening treatment. Some of your clients may be using home whitening products or toothpaste which may limit results as part of the process has

already been done.

Q Can I use a stronger gel?

A We offer the strongest gels available for use outside of a dental clinic in Australia and the highest grade equipment. When the process is done correctly most clients achieve maximum results in one treatment (that's 2/3 x 20 minute sessions) with our products which for the average person is 5-14 shades lighter. What this means is 5-14 shades is the average level of staining a person with an average diet and lifestyle has on the surface of their teeth, who hasn't had a whitening treatment for at least 12 months and hasn't been using home products.

Q What is the maximum result that can be achieved?

A Teeth whitening bleaches the stains on the surface of the teeth, it doesn't whiten enamel no matter who does it, even a dentist. This is the case no matter how strong the lamp is or how strong the peroxide is. Teeth with very little staining due to good diet or the use of home whitening treatments may not show as large a result. The more stained a person's teeth the greater the result will be. The whiter the persons teeth were originally the whiter the result that can be achieved. Each client will have unique results and will have a different start and post treatment shade and will lighten by a different number of shades.

We like to make sure all our clients have the right expectation about teeth whitening so they can pass this on to their clients. Our clients get the best results possible with teeth whitening; however it is important to remember results are very subjective as each individual is different.

Q Want to speed learn how to get the best possible results with our teeth whitening process?

A Here's how to get the best possible results:-

Store your gels in a cool dry place and don't allow gels to get hot at any time i.e. don't leave in the car. This is very important (if gel gets hot it will reduce its potency). Remove the product at least one hour prior to the treatment as cold gel will be less active and therefore less effective.

Ensure you are doing 2-3 x 20 minute sessions with the LED accelerator lamp directly in front of the clients mouth.

The first gel application must be completely removed from the teeth before reapplying the second treatment as any remaining inactive gel will prevent penetration of the fresh gel.

Ensure you apply the gel evenly on the teeth so you have an even coating over every tooth, not



too thin and not too heavy that it creates blobs of gel on teeth.

Make sure the LED accelerator lamp is as close to the clients mouth as possible without touching.

The client must be FULLY reclined during the entire treatment or they will drool and their saliva will dilute the gel from their teeth.

It is important to ensure that during the process the clients head remains straight and directly under the lamp as using a mobile phone or device will often result in the client moving their head and is best avoided.

Make sure the teeth are clean and dry with the oral wipe prior to the gel application.

Check the client has covered all the teeth to the gum line and in between the teeth.

To monitor results with the shade guide ensure you are taking before and after photos in the same light and at the same angle.

Check if your clients have been using any home treatments and/or if they have had a recent in-chair treatment as this will reduce the results.

Teeth whitening will only bleach stains on the surface of the teeth it will have no effect on intrinsic stains like dark shadows, dead nerves and tetracycline stains. Veneers and crowns will not whiten.

## ALLERGIES/REACTIONS/SENSITIVITY

Q What if I have a client that is allergic reaction can they use a non-peroxide whitening treatment?

A We would not recommend it as most bleaching agents break down in to elements including peroxide as they work.

Q What happens if a client has a reaction to the treatment?

A We have never had a serious reaction from our gels. The strength of the peroxide is within legal limits for non dental practitioners. If a client does have a reaction it is unlikely to be serious. In the unlikely event treatment is required our labelling is compliant with regulations which include a phone number for the Poisons Information Centre - 24 Hour Telephone Advice Line 13 11 26. Your starter kit includes a client consent form which combined with your insurance offers you protection from litigation.

Q What would you recommend for clients that have sensitive teeth?

A Clients who have sensitive teeth and know in advance have a couple of options to minimise the potential discomfort. You can suggest they use Sensodine toothpaste for a few days to a week before their treatment and for a few days after. You can also offer a non peroxide treatment which is gentler for these clients.

Q I have had clients get blisters inside their lips after the treatment, what can I do?

A Blisters can be avoided by ensuring the gel doesn't come in contact with the soft tissue of the mouth. We have found blister mainly occur in people with lip filler. We are not sure if this is due to a reaction with the products used in the filler or if it is simply because the lips are larger and more likely to come in contact with the gel on the teeth. When blisters do occur they usually heal in 2-3 days. Here are things to check to avoid blisters or sensitivity during the teeth whitening treatment:-

When the client applies the gel ensure they avoid the lips and gums. If they do get some on there have them remove it with a cotton tip.

Make sure the cheek retractor being used is large enough to keep the lips from touching the gel on the teeth.

Always make sure the client is fully reclined. If they are not fully reclined they can drool and this can wash the gel onto the soft tissue.

Make sure the client rinses thoroughly and brushes away any remaining gel with a tooth brush (no toothpaste) to ensure no gel is left in the mouth.

At the end of the treatment get the client to apply the vitamin e to the inner and outer area of their lips and gums.

Q What can I do to minimise blisters from occurring?

A There are a few things which can cause blisters generally the most common cause is getting gel on the lip. This can be avoided by using the correct size cheek retractor. If the clients lip is overhanging the teeth you may need to use a larger size. If any gel does get on the lip please ensure it is removed prior to activating the lamp.

The client attempting to talk with the cheek retractor in their mouth can also be problematic as this can cause the gel to get onto the lips causing a blister. Talking during the treatment will also

stimulate the glands in the cheeks and increase the saliva production which will dilute the gel. Colder weather and wind can also increase the incidents of blisters as clients often have chapped lips prior to the treatment.

The other causes of blisters can be from having lip tattoo or lip filler within 2 weeks prior to the treatment.

## CHEEK RETRACTORS

**Q** Some clients complain cheek retractors are uncomfortable. Also sometimes my clients top or bottom lip hangs over their teeth, what can I do about that?

**A** The cheek retractors can be uncomfortable but there are options to help. We provide a free small and large cheek retractor with kit orders of five or more. Use largest size possible to avoid the lip covering the teeth. If this occurs try a larger cheek retractor. If the cheek retractor is too tight or difficult to insert into the mouth try a smaller size. Silicone mouth trays are also available which are more comfortable and easier to insert but can reduce the results slightly.

## FLUOROSIS

**Q** Are fluorosis spots caused by teeth whitening and how long will it last?

**A** The client will have already had fluorosis teeth whitening will not cause this. The colour variation will always fade, generally in a few hours and it is unlikely it will last longer than 24 hours.

## GEL APPLICATION

**Q** Will my clients find it strange that they apply the gel?  
What will I say if they really want me to apply it?

**A** This is the process with all non dentists; just tell your client it is easier than painting their fingernails. It is simple to do and they can take their time. Unless you are a dentist you are not permitted to touch your client's mouth and you can not apply gel to their teeth.

**Q** Can the left over whitening gel pen be offered to the client as a home treatment?

**A** If they purchase a home kit you could give them the remaining gel but ideally it is better to

throw it away if they don't as a mouth guard is required to protect the soft tissue of the mouth.

**Q** How much gel do I apply to each tooth to get the best result? Should I be aiming to put a thin layer on each tooth or quite a thick layer as there is enough in each pen/syringe application for a decent layer?

**A** To get the best result you want an even layer of gel, not too thin but not so thick there are blobs otherwise the gel will drip on the soft tissue of the mouth. There is plenty of gel in the pen for 3 applications.

**Q** Is the gel to be removed between each 20 minute treatment, is this just done with the finger wipe or does the clients wash their mouths out with water?

Generally rinse with water and use the oral wipe to remove the rest. If necessary the client can use a tissue. It depends a little on the facilities you have available.

**Q** What is the best gel application? Pen or prefilled mouth tray?

The paint on peroxide pack with the gel pen is our most popular and effective treatment and will give your client the best possible results. Each pack contains everything you need to treat one client for a triple treatment. The packs are for single use on one client and all items are disposable so there is no sterilising for you to do or risk of cross infection.

The prefilled mouth tray contains non-peroxide gel and is used for clients that have sensitivity issues. There is one tray in each pack.

There are 10 treatments in the starter system. You are welcome to purchase more if you wish they range from \$16.95 each.

**Q** How long does hydrogen peroxide gel last and how do I store it?

**A** In general the shelf life of gels is around 12-24 months if stored correctly. We recommend storing your gels under 25 degrees to maximise the shelf life. Store in a cool dry area away from sunlight.

Refrigerating your gels will increase their longevity. If refrigerated remember to take the gels out of the fridge one hour before a treatment so that the gels come back to room temperature.

## TRAINING AND QUALIFICATIONS

Q Do you offer teeth whitening training and a certificate? Is it nationally recognised?

A When you order with us we provide you with access to our online training so you can be ready when your equipment arrives. We are here to support you with teeth whitening information, training, product knowledge and experience.

We provide you with a certificate and our training is nationally recognised by insurance companies such as Insurance Placement Solutions however no teeth whitening training is officially government accredited.

You will receive information on current regulations and insurance as well as comprehensive online training. We provide printable client consent forms, fact sheets and after care sheets. We also keep you updated with the latest client FAQ's and offer social medial promotional material to help you get the word out about your new service.

Q If I am asked what qualifications I have to offer teeth whitening, how do I answer?

A There are no official credentials for teeth whitening in Australia. Generally dentists receive vendor training just like you. Let your client know you have done a training course in self administered cosmetic teeth whitening and are insured to provide the procedure. (Make sure you have your insurance up to date at all times).

## TREATMENT TIMES

Q How many times per year can a client have an in-chair teeth whitening treatment?

A The ADIA recommend having a teeth whitening treatment up to 3 times a year. Performed this often however you may only need one 20 minute session for maximum results. The results achieved will not be as impressive and could be as little as one or two shades when done this regularly.

Q My client wants their teeth whiter, when can they have another teeth whitening treatment?

A There is no exact times given for the time in between a treatment, however about 3-4 months is as often as would be necessary. These times are a recommendation and not a rule so you can

be a little flexible.

Keep in mind your clients teeth will only whiten to the colour they were originally grown. Teeth whitening is a stain removal process and results are subjective as each client is different and so is their lifestyle. Your client will not necessarily be able to get the tooth colour they want as this may not be achievable with whitening. The paper white teeth seen on TV are actually veneers not natural teeth.

Q Why is peroxide whitening treatment done in 2 x 20 minute sessions instead of all at once?

A After about 20 minutes the peroxide gel 'peaks'. This means it has completed the reaction for that application and very little stain breakdown will occur from this point. This is why it is important to completely remove one application prior to applying the next one.

Q If you offer a third 20 minute session will the client's teeth lighten anymore?

A Teeth will only become lighter to the colour they were originally grown. With some clients a 3rd 20 minute session will whiten further but not always. This will only improve the results if there are still stains on the surface of the teeth.

Q What if the gel is left on longer than 20 minutes?

A Leaving it on slightly longer won't hurt but it is of little benefit. Keep in mind the cheek retractors are not very comfortable for your client so the less time they have to wear it the better. Leaving the gel much longer than necessary isn't just ineffectual but it can cause aggravation both to teeth and gums.

Q How often should my client come back for a maintenance treatment?

A Every 3-4 months is all that is required for maintenance. A top up treatment will only be beneficial if the client still has staining visible after the triple treatment.

## MOBILE CHAIRS

Q What colour are the mobile chairs?

A The mobile chairs *only* come in the colours displayed on our website.

## COST OF TEETH WHITENING

Q How much should I charge?

A Yes, \$150 – \$300 per session is the going rate. Check what your competitors are charging in your area and look to charge mid range between the middle and top end price. Around the 70% price point brings in the best quality clients. Pricing too cheap will only attract the sort of clients who are looking for whatever is cheap and free. These clients rarely become regular clients and are generally the type of client you don't want.

This page has been intentionally left blank



## Appendices

### APPENDIX 1 – TROUBLESHOOTING GUIDE

FAULT TYPE	FAULT REASON	SOLUTION
No display	Switched off	Turn on
No output	Settings not correct	Input relevant settings
Low luminosity	Only a single LED setting has been used (A or B)	Select both A and B
Low luminosity	Power not set at 100%	Set to 100% and reduce to 80% after 2 client treatments.

### APPENDIX 2 – TAKING CARE OF YOUR LAMP

1. Avoid getting hair, fibres and dirt in the LED light. We recommend covering the LED lights when NOT in use.
2. If something unexpected happens when using the lamp please contact your supplier.
3. Your equipment has been tested and approved under Australian standards. Please do not disassemble the light head or the control system without speaking with one of our staff or consult your electrical engineer.

### APPENDIX 3 – TEETH WHITENING PROCEDURE GUIDE

1. Clean your teeth 30 minutes to one hour prior to your appointment. **Brush very gently to avoid gum irritation.**
2. Begin by reading and signing the disclaimer.
3. Select your current tooth shade based on your darkest tooth with your technician.
4. We will then provide you with your professional whitening kit containing-  
3 x Finger Wipes, 1 x Vitamin E Swab, 1 x Applicator Gel Pen, 1 x Cheek Retractor, 2 x Bib.
5. Place bib on mat absorbent side up..
6. Insert Cheek retractor into mouth ensuring that the curved flanges go inside the cheeks with the arch facing up like a smile and in front of your chin, see below.
7. Rub the front teeth with finger wipe, with blue side towards the teeth. Discard into your waste bag when finished.
8. Twist the handle on the Applicator pen until a bead of gel extrudes onto the brush tip. Using a hand held mirror, dab the bead of gel onto one tooth at a time and spread the gel evenly as close as possible to the gum line but avoiding the gum. Repeat until all visible upper, lower and between teeth are completely covered.
9. Put on the safety glasses and lie in RECLINED position and relax for 20 minutes while the led light works its magic. If pain or irritation occurs rinse immediately.
10. Remove Cheek retractor and rinse/wipe mouth.
11. Repeat steps 5-10 three times (3 sessions) for an initial client treatment and two times (2 sessions) for a maintenance treatment.
12. Use vitamin E swab after treatment if gum irritation occurs, snap head at red line, allow liquid to drain into the opposite head of the swab and rub over gums.
13. Select your new whiter brighter shade and enjoy your new smile!

*Any white blotches on teeth following the procedure will disappear within 1-24 hours. Whitening will not remove Tetracycline stains. Following your whitening treatment avoid staining beverages as much as possible, including coffee, tea, red wine, colas, and citrus beverages. If you must consume these, it is highly recommended to do so through a straw.*

